

## 5. CHEMPROTECT TRAINING AUTOINJECTORS

### 5.1. TRAINING AUTOINJECTOR DOUBLEPEN OA

- A perfect replica of DOUBLEPEN OA with the exact triggering mechanism
- Metal stick replacing safely the needle
- Can be used minimum 200 times




#### 5.1.1. LABELLING



#### 5.1.2. USAGE OF TRAINING AUTOINJECTOR DOUBLEPEN OA

1. Remove metal safety pin, place autoinjector against thigh muscle
2. Push safety fuse for antidote application and hold 10 seconds
3. Remove the autoinjector from the muscle
4. Push discharged autoinjector against hard surface and simultaneously move safety to the default position.
5. Insert cotter pin back inside the safety fuse.



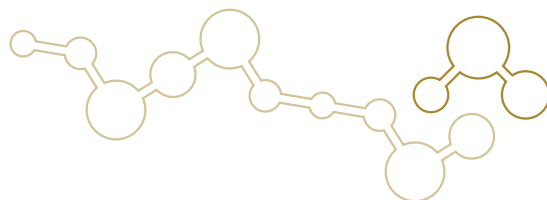






### 5.1.3. MANUAL

**CHEMPROTECT<sup>SK</sup>**

**INSTRUCTIONS FOR USE – TRAINING AUTOINJECTOR**

### DOUBLEPEN OA

- A training autoinjector is designed for training the use of autoinjector Doublepen OA which is produced by ChemProtect.SK.
- A training autoinjector does not contain any active ingredient or needle. Its correct use is safe and simple.
- Use only as instructed.
- Do not use against sensitive parts of the body, eyes, etc. Can cause injuries!

- 1)  Remove cotter pin, place autoinjector against thigh muscle (femoral or gluteal muscle).
- 2)  Push safety fuse for „antidote application“ and hold 10 seconds.
- 3)  Remove autoinjector from muscle.
- 4)  Push discharged autoinjector against hard surface and simultaneously move safety fuse to the second (default) position.
- 5)  Insert cotter pin back inside safety fuse. Training autoinjector is ready for another use.

[WWW.CHEMPROTECT.SK](http://WWW.CHEMPROTECT.SK)

### 5.1.4. PACKAGING

Offered in green and brown pouch.

